



Software	Mode	Time	Key 1	Key 2	Key 3	Notes
Apteryx XVLite or XRayVision	Sequential Mode	50ms	SPACE	n/a	n/a	Includes other OEM versions of XV and XVLite
Choice Imaging or CADI	Sequential Mode	50ms	G	n/a	n/a	
Dentrix Imaging	Send All Mode	50ms	ALT+C	ALT+S	n/a	Choose any USB camera in preferences
Dexis	Send All Mode	50ms	T	SPACE	n/a	
EagleSoft or Patterson	Send All Mode	50ms	F	S	n/a	
EasyDental Imaging	Send All Mode	50ms	ALT+C	ALT+S	n/a	Choose any USB camera in preferences
Kodak Dental Imaging	Sequential Mode	50ms	F5	n/a	n/a	First button press freezes image, second press saves image and goes live again.
VixWin Platinum	Send All Mode	50ms	SPACE	ALT+G	SPACE	
VixWin Pro	Send All Mode	50ms	ALT+F	ALT+G	ALT+F	
XDR	Sequential Mode	50ms	SPACE	n/a	n/a	First button press freezes image, second press saves image and goes live again.