



Updated 9/2018

Camera Button Utility Chart

Software	Mode	Time	Key 1	Key 2	Key 3	Notes
Adstra	Send All Mode	50ms	SPACE			
Apteryx XV Lite or XRayVision	Send All Mode	50ms	SPACE			Includes other OEM versions of XV and XV Lite Only needed for QuickCam Duo and HD
ATECO	Send All Mode	50ms	SPACE			
Cleardent	Send All Mode	50ms	SPACE			
Curve Hero	Send All Mode	50ms	ENTER			
Dentrix Image	Send All Mode	50ms	ALT+C	ALT+S		Must be double clicked
Dexis	Send All Mode	50ms	T	SPACE		Dexis 10 only requires Key 1 to be set.
EagleSoft	Send All Mode	50ms	F	S		
EasyDental Imaging	Send All Mode	50ms	ALT+C	ALT+S		
Kodak Dental Imaging	Send All Mode	50ms	F5	F5		
Mogo	Send All Mode	50ms	F8			
Romexis	Send All Mode	50ms	ALT+F	ALT+S		
Schick CDR Dicom	Send All Mode	50ms	ALT+C	U		
SoftDent Imaging	Send All Mode	50ms	F5	F5		ICam works, QC Duo works with SetupSTV
SOPRO	Send All Mode	50ms	ALT+F	ALT+F		
TigerView	Send All Mode	50ms	CTRL+F	CTRL+S		
VixWin Platinum	Send All Mode	50ms	SPACE	ALT+G	SPACE	
VixWin Pro	Send All Mode	50ms	ALT+F	ALT+G	ALT+F	
XDR	Send All Mode	50ms	CTRL+I	CTRL+I		Might not need MDC2.1 for ICam if it captures 2x just close it.